

Quick Reference Guide for EV29L Reusable Transrectal Needle Guide/Plug

1. Reprocessing Before First and Every use

- a. Disassemble before reprocessing
- b. Cleaning Cidezyme, 5 minutes
 - Use an appropriate brush for channels/cannulas
 - Flush all channels/cannulas with cleaner
- c. Sterilization
 - Sterilization: Class B autoclave
 - 134°C (273°F) for 3 min
 - 132°C (270°F) for 4 min
 - Alternately (if autoclave is not available) High-Level Disinfection: Cidex OPA, 12 minutes
 - Flush all channels/cannulas with disinfectant
- d. Inspection after reprocessing

2. Assembly/Disassembly*




3. Attaching to the EV29L Transducer

4. Software Adjustments

- a. Anesthesia Mode: Select **35°** option for needle guide angle
- b. Biopsy Mode: No change required

*This step is not required for use as plug (for imaging only procedures).


**Install the plastic enclosure underneath the sheath for use as plug (for imaging-only procedures).

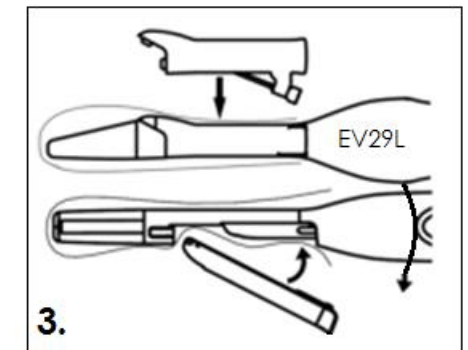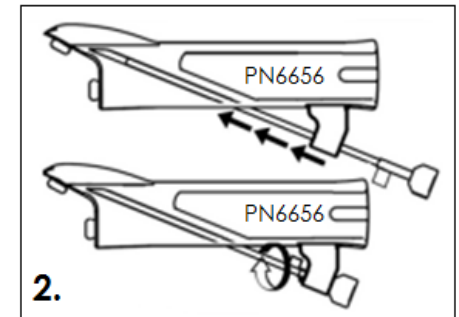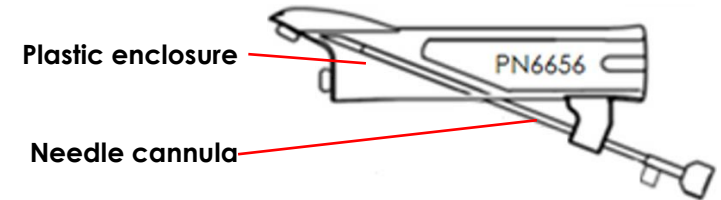

Refer to the following Manuals for Detailed Instructions:

Step	Manual	Section of Manual
1. Reprocessing (before first and every use)	Care, Cleaning and Use Guide for EV29L	Chapter 4: Reprocessing Section 3: Reprocessing the EV29L Reusable Transrectal Needle Guide
2. Assembly/ Disassembly		Chapter 2: General Information Section 2.2.2.3: Preparing the Needle Guide for Use
3. Attaching to the EV29L Transducer		Chapter 3: Preparing for Imaging Section 2.1.1: Attaching the Transrectal Needle Guide
4. Software Adjustments	ExactVu Operation and Safety Manual	Chapter 5: Using 2D Mode Section 2.3.1: Using the Needle Guide Overlay with the EV29L Transducer

If you require **technical assistance** with your ExactVu™ micro-ultrasound system or with the EV29L Reusable Transrectal Needle Guide, contact Technical Support using the contact information for your region at <https://www.exactimaging.com/contact-us>.

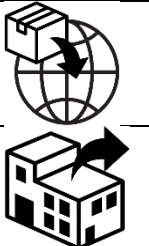 Exact Imaging Inc. 7676 Woodbine Avenue, Unit 15 Markham, ON L3R 2N2, Canada +1.905.415.0030 info@exactimaging.com				
EU REP	Emergo Europe Westervoortsedijk 60 6827 AT Arnhem The Netherlands	CH REP	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland	UK Responsible Person Emergo Consulting (UK) Limited c/o Cr360 – UL International Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom
	Exact Imaging BVBA Offergemsesteenweg-Zuid 808 / b508 9000 Gent Belgium	Exact Imaging Catalog Numbers • EV-29L: EV29L Transducer • EV-BIOGR: EV29L Reusable Transrectal Needle Guide, 18 GA • EV-BIOG-R16: EV29L Reusable Transrectal Needle Guide, 16 GA • EV-SYS-220: ExactVu™ Micro-Ultrasound Imaging System (220V) • EV-SYS-120: ExactVu™ Micro-Ultrasound Imaging System (120V) • EV-SYS-100: ExactVu™ Micro-Ultrasound Imaging System (100V)		