

Approved Chemicals List for ExactVu™ Transducers

Type of Chemical	Chemical Name	Manufacturer	Transrectal Transducer			Surface Transducer	Validated Contact Time
			EV29L	EV9C SN 6750-	EV9C SN 6087-	EV5C	
Cleaner	CIDEZYME®	Advanced Sterilization Products	✓	✓	✓	×	1 minute
	ENZOL®	Advanced Sterilization Products	✓	✓	✓	×	1 minute
High-level Disinfectant	CIDEX® OPA	Advanced Sterilization Products	✓	✓	✓	✓	12 minutes
	CIDEX®	Advanced Sterilization Products	✓	✓	✓	×	45 minutes
	Revital-Ox™ Resert®	Steris Corporation	✓	✓	×	✓	8 minutes
	UltrOx™	CIVCO Medical Solutions	✓	✓	×	✓	8 minutes
Wipe/Spray	Transeptic® Spray	Parker Laboratories	×	×	×	✓	30 seconds
	Tristel Trio	Tristel Solutions	✓	✓	✓	✓	30 seconds (sporicidal)

* FDA has accepted "Metricide Activated Dialdehyde Solution" manufactured by Metrex Research, Inc. (2.6% glutaraldehyde, 14 day re-use) as identical to Cidex®.

Table 1: Approved Chemicals List

The images below show the soak level for the transducers.



Figure 1: EV29L Soak Level

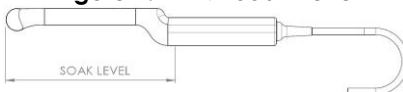


Figure 2: EV9C Soak Level



Figure 3: EV5C Soak Level

CAUTION

EN-C25



Do not immerse the transducer beyond its soak level.

NOTE




Use cleaning and disinfection products which are approved for use in your region.

Follow the manufacturer's instructions for use for disinfectants and cleaners.

Please refer to the *Care, Cleaning, and Use Guide* for your transducer for instructions on reprocessing it between uses, and the location of its soak level.



Exact Imaging Inc.
7676 Woodbine Avenue, Unit 15
Markham, ON L3R 2N2, Canada
+1.905.415.0030
info@exactimaging.com

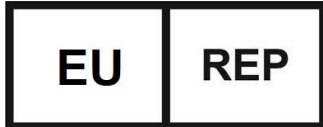

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland



Exact Imaging BVBA
Ottergemsesteenweg-Zuid 808 / b508
9000 Gent
Belgium



UK Responsible
Person

**Emergo Consulting (UK) Limited c/o Cr360 – UL
International**
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

Trademarks

Exact Imaging trademarks:

- ExactVu™

CIDEZYME®, CIDEX® and ENZOL® are registered trademarks of Johnson & Johnson.

Revital-Ox™ is a trademark of Steris Corporation. Resert® is a registered trademark of Steris Corporation.

Transeptic® is a registered trademark of Parker Laboratories Inc.

UltrOx™ is a trademark of CIVCO Medical Solutions.

Exact Imaging Catalog Numbers

- EV-29L: EV29L Transducer
- EV-9C: EV9C Transducer
- EV-5C: EV5C Transducer
- EV-SYS-220: ExactVu™ Micro-Ultrasound Imaging System (220V)
- EV-SYS-120: ExactVu™ Micro-Ultrasound Imaging System (120V)
- EV-SYS-100: ExactVu™ Micro-Ultrasound Imaging System (100V)

Version information

System: ExactVu™ High Resolution Micro-Ultrasound System

Approved Chemicals List for ExactVu™ Transducers, Revision 2.1, *original instructions*

