

Quick Reference Guide for EV29L Non-Sterile Reusable PrecisionPoint Adapter

1. Reprocessing Before First and Every use

a. Cleaning Cidezyme, 5 minutes

- Use an appropriate brush for channels/cannulas
- Flush all channels/cannulas with cleaner
- Rinse for 5 minutes following cleaning.

b. Sterilization


• Sterilization: Class B autoclave

- 134°C (273°F) for 3 min
- 132°C (270°F) for 4 min

c. Inspection after reprocessing (no visible cracks, discoloration or damage)

2. Attaching to the EV29L Transducer


Install the plastic adapter on top of the sheath. Attach each arm of the PrecisionPoint guide to the sides of the adapter. The PrecisionPoint guide is single use and should be used according to the manufacturer's IFU.

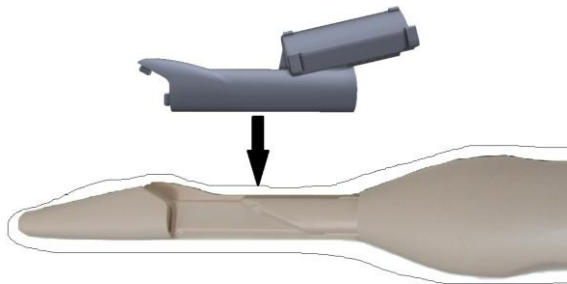

2a



2b

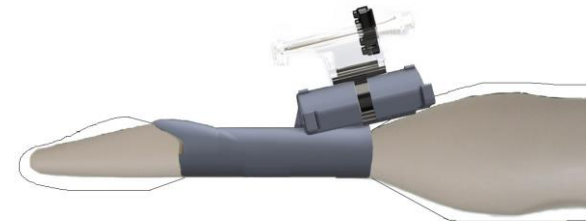

2c

3. Working Lifetime

When used with proper care, the EV29L Non-Sterile Reusable PrecisionPoint Adapter is designed for a working lifetime of whichever occurs first of 3 months or 75 reprocessing cycles.

Refer to the following Manuals for Detailed Instructions:

Step	Manual	Section of Manual
1. Reprocessing (before first and every use)	Care, Cleaning and Use Guide for EV29L	Chapter 4: Reprocessing Section 3.2, 3.3 and 3.5: Reprocessing Reusable Needle Guides and Accessories* * The Cleaning, Sterilizing and Inspection instructions are also applicable to the EV29L Reusable PrecisionPoint Adapter.
2. Attaching to the EV29L Transducer	Care, Cleaning and Use Guide for EV29L	Chapter 3: Preparing for Imaging Section 2.1.1: Attaching the Needle Guide* * These instructions are also applicable to the EV29L Reusable PrecisionPoint Adapter.

If you require **technical assistance** with your ExactVu™ micro-ultrasound system or with the EV29L Reusable PrecisionPoint Adapter, contact Exact Imaging Technical Support. Refer to <https://www.exactimaging.com/contact-us> to find the contact information for your region.



Exact Imaging Inc.
7676 Woodbine Avenue, Unit 15
Markham, ON L3R 2N2, Canada
+1.905.415.0030
info@exactimaging.com



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

UK Responsible Person
Emergo Consulting (UK) Limited c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ United Kingdom



Exact Imaging BVBA
Ottergemsesteenweg-Zuid 808 / b508
9000 Gent
Belgium

Exact Imaging Catalog Numbers

- EV-29L: EV29L Transducer
- EV-BIOGR-PP: EV29L Reusable PrecisionPoint Adapter
- EV-SYS-220: ExactVu™ Micro-Ultrasound Imaging System (220V)
- EV-SYS-120: ExactVu™ Micro-Ultrasound Imaging System (120V)
- EV-SYS-100: ExactVu™ Micro-Ultrasound Imaging System (100V)